

## DETECTO HAS A WEIGH WITH BABIES

### Model 6730 Infant Scale



*Years of experience have gone into creating the latest in accurate, dependable infant scales... DETECTO's 6730. Designed to be used with utmost ease, this scale can weigh even the most active baby. With step-by-step instructions right on the keypad, there's never any question about what to do next.*

- **Battery Operation**
- **Long-lasting, Safety-styled Baby Tray**
- **Lock-in Weight Display**
- **Color-coded Keypad with Instructions**
- **Keypad-selectable Pounds/Ounces or Kilograms/Grams**
- **Standard Serial Outport for Use with Computer or Printer**
- **Easy Cleanup**



# So Easy to Use - You'll Never Get Rattled!



With all there is to do for a newborn, weighing that new little bundle shouldn't be a difficult task. DETECTO has made it as easy as can be with the 6730 Infant Scale. Plastic tray accepts baby comfortably and safely. Color-coded keypad features large "on/off", "zero" and "lb/kg" buttons, plus step-by-step instructions, so even a novice can operate the scale with ease.

The 6730 is extremely portable. It operates on "D"-size alkaline or Ni-Cad batteries. The scale has a lock-in weight feature which compensates for typical newborn movement. When weight stabilizes, it is "locked in" and displayed; infant can be removed and its weight retained until cleared. Special filtering within the microprocessing system minimizes the effects of movement on the scale, and the final weight is determined quickly.

DETECTO's 6730 weighs in pounds and ounces or kilograms and grams - you decide which by the touch of a button. (Give the parents the weight in pounds and ounces; record or chart the weight in metric units.) Seamless plastic tray and sealed keypad enable the scale to be cleaned and sanitized easily. Automatic turn-off maximizes battery life, and an added plus is the ability to retrieve the last scale weight (in case you didn't record it before the scale turned off).

The 6730 also is equipped with a standard serial output so it can be used with a computer or printer to store and record weights. Consider the 6730 Infant Scale for hospital, physician's office, clinic or home. It has the time-proven DETECTO combination of ease of use, accuracy, speed, versatility and dependability.



Step-by-step operation instructions are always in view on scale keypad.



P.O. Box 151, Webb City, MO 64870 USA  
Ph: 417-673-4631 or 800-641-2008 • Fax: 417-673-5001  
Visit our Web site at <http://www.cardet.com>  
e mail: [cardinal@cardet.com](mailto:cardinal@cardet.com)

©Copyright 1995 Cardinal Scale Mfg. Co. • Printed in USA • C146A



Active babies are no problem on DETECTO's 6730 with lock-in weight display. Scale's 1" high LCD's are easy to read.



Optional label printer (Model P200) provides documentation of baby's weight, for transfer to medical record.

## SPECIFICATIONS

MODEL	6730
Capacity	30 lb x 2 oz, 15 kg x 5 g
Display	5 digits, 1" high LCD's
Scale Dimensions	23"W x 14"D x 7"H
Tray Dimensions	25.75"W x 12"D at top, tapering to 25.75"W x 9.25" D at bottom
Tray Material	White molded plastic
Power Requirements	Operates on 8 "D"-size alkaline batteries or 8 "D"-size Ni-Cad batteries
Status Indicators	Pounds/Ounces, Kilograms/Grams, Weight Lock, Low Battery
Controls	On/Off, Zero (used to return to zero or as tare to offset weight of blanket or cover), Lb/Kg
Operating Temperature	0° to 40°C (32° to 104°F)
Shipping Weight	25 lb
Weight Sensor	Strain-gauge load cell
Motion Detector	Senses motion on the scale and locks in weight display after motion has ceased, or after 5 seconds
Color	White front panel with blue accenting, white tray
Options	P200 thermal label printer with serial connection cable

Cardinal Scale reserves the right to improve, enhance and modify features and specifications without prior notice.

**Sold By:**